
Internet: www.basko.com
 E-mail: sales@basko.com
 Fax: +31 (0) 75 - 612 63 73
 Phone: +31 (0) 75 - 613 15 13
 NL-1505 HX Zaandam
 Pieter Liefthoekweg 16

Basko Healthcare

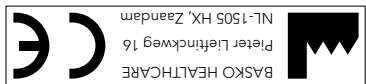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Instructions for Use

Your doctor will prescribe a P.E.A.care 2.0 Splint and determines the duration of treatment. The medical instructions on daily use must be followed exactly. The technician fitting the Splint is responsible for the correct fitting and functioning. In the event of skin irritation, abnormal pain or any other complication you believe to be related to the product, please contact the your doctor and/or technician.

Intended use / Indications

The use of a P.E.A.care Splint is indicated for the treatment of club foot inter alia according to the Ponseti method. The P.E.A.care Splint is an orthotic device. The complete orthotic device consists of a P.E.A.care Splint with Clubycare Orthotic Boots attached to its ends. When using the orthosis, the feet will be kept in the necessary external rotation position and at shoulder width. This will retain the foot correction achieved during preliminary treatment and help prevent relapse.

Application is indicated for:

- Arthrogryposis multiplex congenita (ICD10: Q74.3)
- Pes equinovarus congenitus (ICD10: Q66.0)

Permitted conditions of use /Application Points

Brace may not be used in water.

Risks associated with use and contraindications

There are no known risks associated with use and no contraindications.

Maintenance Instructions

Check the braces regularly for wear around the fittings and quick release mechanism.

Information on Materials Used

IXEF /POM plastics, nickel-plated brass and stainless steel

Cleaning/Disinfection Instructions

Clean the braces with a cloth and warm water to a max. 40°C. Do not dry-clean. Not suitable for washing machine. Do not tumble dry.

Fitting Instructions for P.E.A.care 2.0

First check the content of the package:

- 1) P.E.A.care 2.0 Splint
- 2) Allen key (1x) and flathead screws (4x)
- 3) Torque key (1x)
- 4) Technical fitting instructions
- 5) Information for parents/caregivers

Assembling the Splint

Step 1: Measure the child's shoulder width (Fig.1) and determine the correct Splint size (see size chart).

Step 2: Setting the Splint width at shoulder width: Loosen the set screws with the Allen key (Fig.2a) and set the Splint width equal to shoulder width (centre to centre) (Fig.2b). Position the set screws as far from each other as possible and tighten the screws.

Step 3: Setting the Abduction Angle (0° to 80°): Squeeze the two red release latches (Quick Release) and slide the foot plate away from the foot plate connector (Afb.3). Use the Torque key to unscrew the set screw on top of the connector (until it blocks) (Fig.4). At the bottom of the Splint is a protractor. Tilt the connector slightly, turn and position it into the desired abduction angle (in 5° increments) using the marker on the side of the connector as a reference (Fig.5). Let go of the connector by which it locks into the set position.

Step 4: Setting the Dorsal Flexion Angle (0° to 20°): Next to the setscrew on top of the connector a second protractor is located. If a Dorsal Flexion setting is desired (according Ponseti), tilt the connector backwards to the desired Dorsal Flexion position (in 5° increments) and use the marker next to the protractor as a reference (Afb.6).

Step 5: Tighten the connector setscrew.










Step 6: Repeat steps 3 till 5 on the other side of the Splint.

Step 7: Assemble the footplate's with the supplied flathead screws onto the sole of the (Clubycare) shoe (Afb.7).

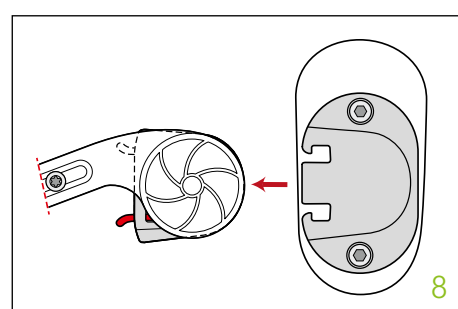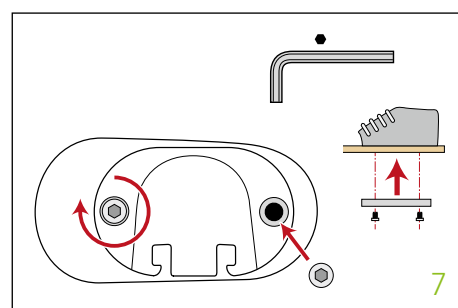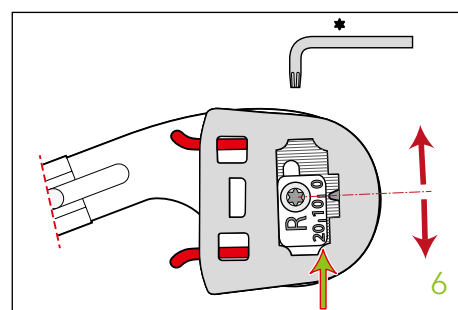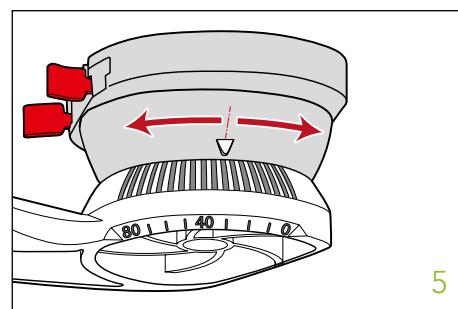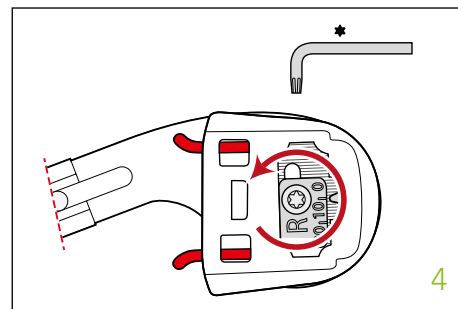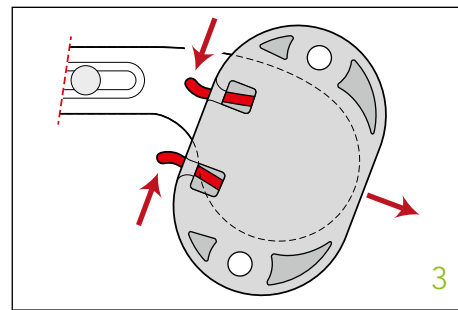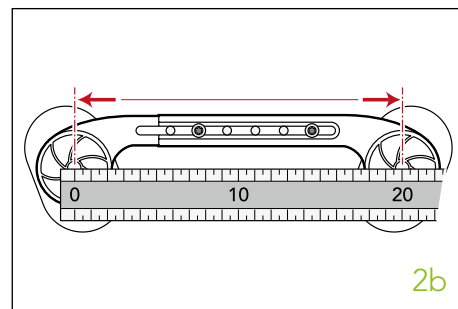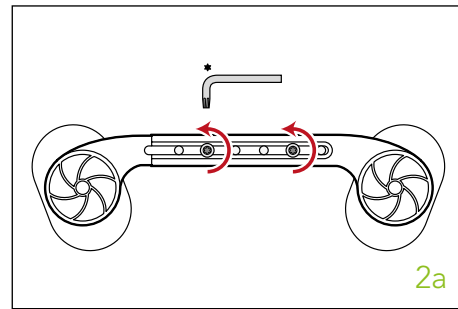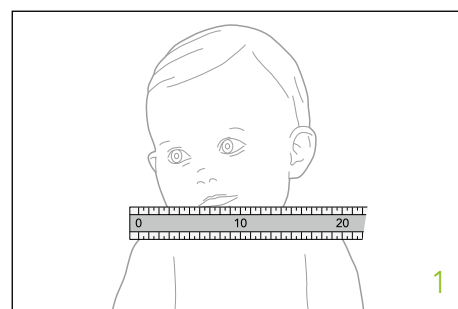Step 8: Check the function of the quick release mechanism. Slide the shoe with the foot plate laterally onto the footplate adapter. Press and click the foot plate onto the adapter (Fig.8). Pay attention both the identification markers (left) and R (right) on the bar and foot plates correspond. To release, squeeze the red releases latches together and push the shoe with the foot plate away from the Splint.

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