







Stomy

Ostomy belt


- NL** Gebruiksaanwijzing Stoma bandage
- DE** Gebrauchsanleitung Stomabandage
- ENG** Instructions for use Ostomy belt
- FR** Mode d'emploi Bandage de stomie
- SE** Användarinstruktioner Stomibandage
- NO** Bruksanvisning Stomibrokkbelte
- DK** Brugerinstruktioner Stomibælte
- FI** Sovitusohjeet / Avanneliivi
- IT** Istruzioni d'uso della fascia
- ES** Instrucciones de uso de las fajas

Basko
Healthcare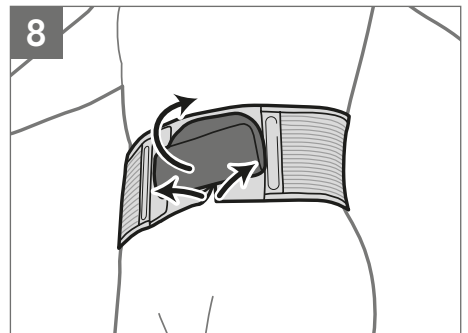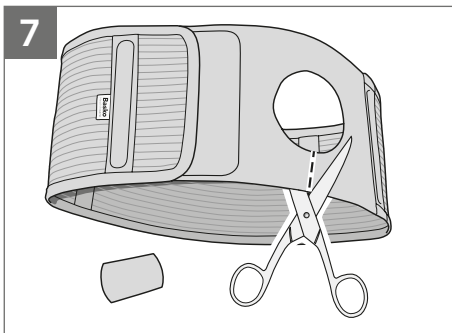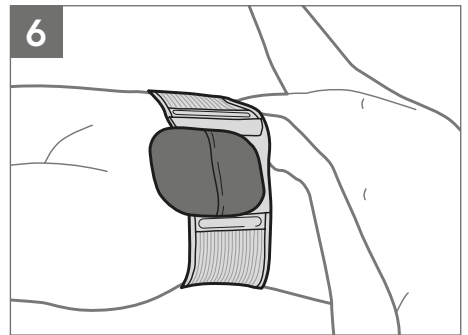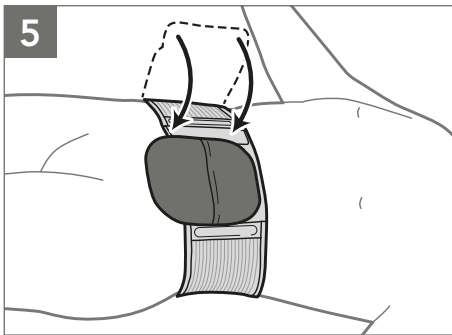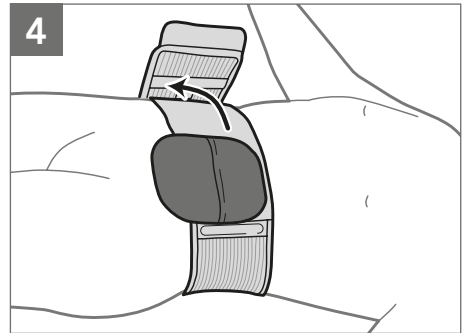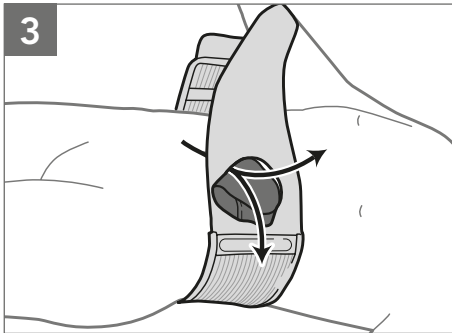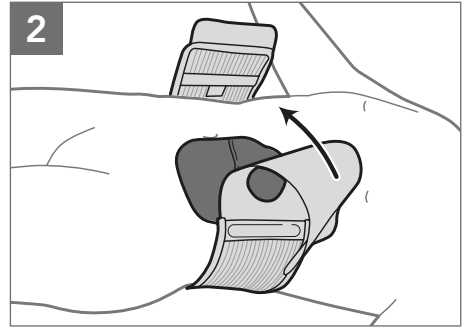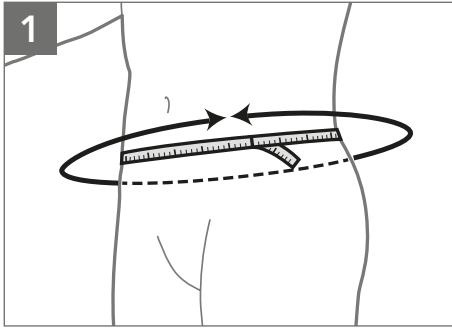

Size chart

Art. nr.	Height	Ø belt aperture
370.15-S	15 cm	Ø 72 mm
370.15-M	15 cm	Ø 72 mm
370.15-L	15 cm	Ø 72 mm
370.15-XL	15 cm	Ø 72 mm
370.15-XXL	15 cm	Ø 72 mm
370.15-XXXL	15 cm	Ø 72 mm
370.15-XXXXL	15 cm	Ø 72 mm
370.20-S	20 cm	Ø 72 mm
370.20-M	20 cm	Ø 72 mm
370.20-L	20 cm	Ø 72 mm
370.20-XL	20 cm	Ø 72 mm
370.20-XXL	20 cm	Ø 72 mm
370.20-XXXL	20 cm	Ø 72 mm
370.20-XXXXL	20 cm	Ø 72 mm

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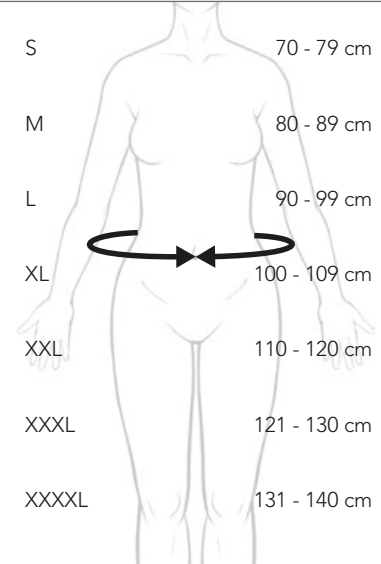Instructions for use

Your ostomy, wound nurse or physician has advised you to start wearing a stoma belt. For optimal care, the user should read these instructions carefully before applying the belt. This product is a medical device and is intended for use by one person only.

Intended use of the product

The Stomy belt is mainly used postoperatively to support the abdominal wall and prevent the formation of hernia and prolapse, for example during physical activities where pressure on the abdominal wall is higher (sports, gardening, lifting, vacuuming, etc.). The Stomy belt offers the wearer a high level of wearing comfort, a safe and secure feeling in daily life.

Applying belt

To wear the Stomy belt correctly, you should preferably apply the belt while lying down. In case of a fracture, in many cases it will sink back into the abdomen and the belt provides counter pressure. This way the fracture is optimally supported. The belt is closed in such a way that you can still slide your hand in between. The belt is then sufficiently tight to provide support but still wears comfortably.

Fig. 1

The circumference of the abdomen must be measured to select the correct size. Use the size chart to determine which size is the best fit.

Fig. 2

Place the ostomy belt on the bed (or put it on loosely) and lie down on it. Position the velour side with the stoma opening over the stoma pouch.

Fig. 3

Fold the bottom of the stoma pouch into a point and feed it from the inside of the belt through the stoma aperture, leaving the sticky side of the pouch on the inside of the belt.

Fig. 4

Make sure that the stoma pouch is flat and without folds and then place the end of the belt over the abdomen.

Fig. 5

Now close the belt to the desired tension by fixing the Velcro hook onto the velour.

Fig. 6

After closing the belt, check whether it is still possible to slide a hand between the belt and the abdomen. If necessary, adjust the tension. After applying you can stand up again.

Fig. 7 (optional)

You can choose to install an additional closure underneath the stoma aperture for easy pouch exchange without having to remove the belt first. To do so, use a sharp pair of scissors to cut the velour panel in a straight line, directly underneath the stoma aperture. The material does not fray.

Fig. 8

The belt is supplied with an additional Velcro strip. Position it over the opening cut, to connect the two parts. To change the pouch, lift it slightly forward and pull one side of the strip to open. Pull the sticky edge of the stoma pouch loose from the skin and remove the pouch along the bottom of the belt. To apply a new stoma bag, follow the instructions as previously described and re-attach the Velcro strip.

Indication

- Abdominal wall weakness (ICD10: M79.9)
- Cicatricial and Stoma hernia (ICD10: K43.9)
- Colon Conduit (ICD10: Z93.9)
- Colostomy (ICD10: Z93.3)
- Enterostomy (ICD10: Z93.4)
- Ileam Conduit (ICD10: Z93.9)
- Ileostomy (ICD10: Z93.2)
- Prolapse (ICD10: K91.4)
- Ureterocutaneostoma (ICD10: Z93.6)
- Urostomy (ICD10: Z93.6)

Contraindications

There are no contraindications known.

Additional product information / area of use

The belt is not suitable for use in salt water.

Existing user risks

No user risks are known. However, the user must regularly check if pressure points or other skin irritations occur and must immediately contact the ostomy, wound nurse or technician if so.

Warning

The belt must not be worn too tight and must be tensioned in such a way that the hand can still be inserted. The bandage should not come into direct contact with open wounds and should be worn on dry skin. The use of lotion or creams is not recommended and should first be discussed with your stoma, wound nurse or physician.

Maintenance advice

If the belt is not worn, it is important to store it with the closures closed. To elongate the life of the belt, it is important that the Velcro straps remain free of dirt, dust etc. to continue to guarantee good adhesion.

Materials used

Polyester, Polyamide, Elastane and Viscose.

Washing instructions

We recommend washing the belt in warm water at 40° C with a mild detergent. To prevent damage to the fabric, the Velcro closures must be closed prior to washing.

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