NAVY™ Xtra

23563

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ENGLISH

23563 NAVY™ XTRA

Carefully read these instructions before use!

Intended Use:

The individually shaped back plate and the corset's reinforced anterior part stabilize the back and may thereby contribute to pain relief. The product is intended for single person use only.

Indications

Fractures, Post-op Spinal stabilization, Post-Op Disc Surgery, Degenerative Disc Diseases, Herniated Disc, Bulging Disc, Spondylolisthesis, Spondylolysis, Facet Syndrome, Spinal Stenosis.

Contraindications

Unstable spine conditions.

Adjustment

This product should only be altered by a medical professional.

How to use the support

Check that you have the correct size. Some conditions can result in swelling that can mean a variation in the size. If this is the first time using a support it is important to start slowly. The support should initially be used for a short period of time on the first day, then increase the use gradually. Do not use the support overnight / while sleeping. There is no standard length of time that the brace should be worn. The length of time varies among patients. Always follow your clinician's specific recommendations.

Warnings

Do not use directly on open wounds. Use only on dry skin. Do not use lotions on your skin. Make sure the straps are not too tight. If any sign of skin irritation oc-

FITTING INSTRUCTIONS

Navy Xtra consist of three parts:

- The shapeable back section in 3D-LITE™ material.
- Cover for back section.
- Elastic support with stabilizing straps and reinforcement panels in front section.

Heat back panel with a heat gun or in an oven at 80°C. Mould rigid back panel over a snug fitting t-shirt, body stocking or stockinette.

Keep moulding the back panel and secure it with a binder. Continue moulding to get the correct shape and contour. If needed the 3D-LITE can be remoulded several times.

Once back panel has cooled and is rigid (approx. 5 - 10 minutes) slide into pocket and attach to orthosis.

Apply orthosis, with rigid back panel to patient making sure it is moulded appropriately and is comfortable to the patient.

The rigid plastic panels on the front are removable and can be adjusted as needed.

curs, remove the support immediately. Check your skin on a daily basis. Do not leave the back panel in a warm car or near a warm radiator as this may alter the shape. Do not use the brace in chlorinated water.

Material Specifications

The product is made from Nylon, Spandex, Polyester. The plastic back section is made from low temperature thermoplastic 3D-LITE. The cover is made from Polyamide and Lycra. This product does not contain Latex. This product is not flame retardant.

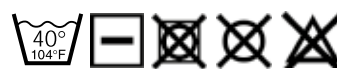
Washing Instruction

Remove the plastic inserts from the front panels and the plastic back section from the pocket. Remove the strip of hook that is used to attach the pocket to the

back support. Do not put any of these parts into the washing machine. The remaining section of back support can be machine washed at a warm temperature (40°C). Dry flat. Do not dry-clean, bleach, tumble dry, or iron. The plastic sections can be cleaned with a mild soap and towel.

Who to Contact

If any problems occur, remove the support and contact your clinician or store where purchased.

USER INSTRUCTIONS

Check that the product is complete and without damage.

Use an undergarment (t-shirt / body stockinette) under the back brace for added comfort. Wash both undergarment and brace on a regular basis.

Loosen the xtra elastic straps. Grasp the front panels, centre back rigid panel along your spine and slide up or down until it rests comfortably in the lower back area.

Pull front panels over the lower abdomen, right and left, then secure. Grasp the outer elastic straps, pull forward, snug the rigid back in position against the spine. Press the hook end into the plush fabric in front. Make sure your abdomen is firmly compressed to 'lift' the abdomen.

For best results make sure the brace is on your lower back and is pulled snug for comfort.

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Camp Scandinavia AB, Karbingatan 38