
Model 2990

Dynamic Hip Abduction Orthosis

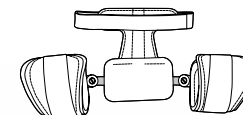Model 2990

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
Working with an orthosis and Hip Dysplasia

Dear Parents / Guardians,


Today your baby received an orthosis. This aid is provided at the age between three until twelve months and must be worn day and night, unless your attending physician instructed you otherwise. The orthosis is not suitable for walking. The orthosis is **mend to be worn** over the cloths.

If the orthosis is fitted, the knees must be able to be flexed for approximately 90 degrees. Usage of the orthosis should not be painful. When your baby cries more than normal after three days, it is the advisable to contact your technician. The following guidelines and images are to clarify the usage of the orthosis. The cooperation of the parents is of decisive interest for the result of the treatment.


Fitting the Orthosis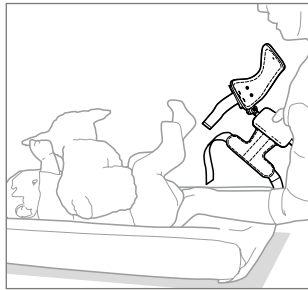


1. Lift the buttocks with the flat hand without pulling the legs and position the orthosis under the lower back.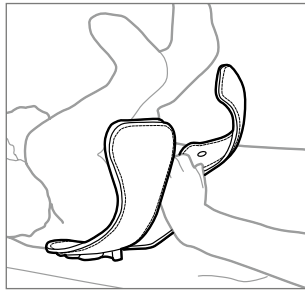


2. The buttocks is aligned to the lower part of the plastic back part of the orthosis.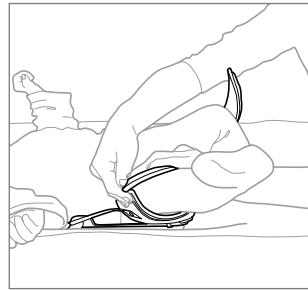

3. First close the cuff to the leg, which is most affected by the dysplasia. Before closing make sure the transparent flap is flattened over the leg.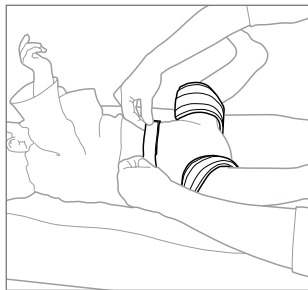

4. Close the Velcro strap leaving just enough room fitting a finger top. Prevent the diaper from getting stuck between the Orthosis. If necessary push it up a little.

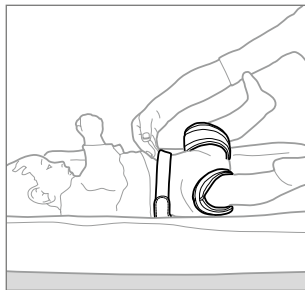5. Close the other cuff followed by the pelvic strap, which provides stability to the orthosis.

6. Support your child holding your hands under the buttocks when lifted. This will prevent poor blood circulation due to pinching of the cuffs.

Tips

- The orthosis may only be removed while changing diapers, during bathing or when your physician instructed you otherwise.
- To maintain save transport for the baby you can stuff the baby seat with a little pillow or towel before seating the baby. It may be necessary to adjust the belt of the seat.
- While changing diapers be aware not to lift or pull the legs or feed of the baby.
- Due to the abducted position of the legs, diaper leakage may occur some more than normal. More frequent changes may be needed.
- The orthosis may be cleaned with water and mild soap. - In case of technical problems we advise you to consult the technician whom provided your baby with the Orthosis.
- With warm weather the cuffs may become dampened. To prevent skin irritation you may create a tricot liner on the inside of the cuffs.
- To prevent the baby from pulling the Velcro it is advisable to have the baby wear oversized tricot short. If this doesn't stop the baby from pulling the Velcro return to your technician and have them change the closure.
- While sleeping the baby will position in a comfortable position to sleep in. It may occur the orthosis to dislocate. Try to reposition the orthosis in its optimum position. Especially during sleeping the hip will recover most and dislocation will pro s pone the effect and may prolong treatment more than necessary.

Cleaning Instructions

Check the Velcro straps on dirt or dust etc. and close all Velcro straps before cleaning, to maintain optimum closure capacity of the Velcro. Clean the orthosis with hand warm water and mild soap.

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