MultiStatic

Correction Joint

Please read these instructions carefully before using the product. Pay particular attention that the safety instructions are followed.

Usage guidelines

- The product may only be fitted by or under the supervision of a competent professional
- Special precautions must always be taken for patients with a visual impairment, cognitive limitations and/or reduced sensibility in the upper and/or lower extremity.
- Changes, modifications and adjustments to the product that are not described in these instructions, are done at the sole responsibility of the person carrying them out.

Intended use of the product

The MultiStatic corrective Joint is used for treatment of correctable contractures, both for children as well as adults. The Joint enables fast, efficient and adequate static treatment of contractures according on the High Load Brief Stretch (HLBS) principles. The angle of the joint is set by an eccentric axis.

The corrective joint is to be used exclusively in orthotics for the upper or lower extremities and should not be weight bearing.

We strongly advise to use the FreeMotion companion joint next to the MultiStatic corrective joint at the contralateral side of the affected joint to ensure sufficient torsional stiffness of the orthosis.

Note:

- No weight bearing
- Fall injury, not suitable for walking

MultiStatic is suitable for static contracture treatment with a neurological or orthopedic origin.

Neurological indications:

- Apoplexy (CVA)
- Cerebral palsy
- Cranial Brain Trauma
- Dystrophies
- Multiple sclerosis
- Status after spinal cord injury
- Spina Bifida

Orthopedic indications:

- Amputation
- Fractures
- Status after ligament rupture
- Status after joint surgery

Contraindications:

- Structural Contractures als:
 - Ankylosis
 - Fibrosis
 - Ossification
- Muscle and bones deformation
- Poor blood circulation

General safety instructions

Risk of injury as a result of incorrect use; the professional should verify correct functioning of the joints prior to delivery of the orthosis to the user. The patient and his/her care giver(s) should also be instructed on donning and doffing of the orthosis and have the joint functionality explained to them. It should also be made clear to them that the spring tension setting should never be changed by anybody else than the professional or therapist, unless the treating physician decides otherwise. The joints should not come into contact with water.

Comment

Specialists opting for and/or fitting and customize this product, should use their professional judgment in choosing the product, fitting it and adequately educating the patient or care giver so as to minimize potential risks for each individual patient. These risks may include the mentioned contraindications, but may also be associated with the unique patient characteristics or circumstances of the patient’s care giver.

ENG Instructions for Use

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FR Instructions d' utilisation

SE Bruksanvisning

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FIN Käsikirja

Art. no.	Unit	Size	Length	Width	Height	Thickness	Weight	Diameter
MS-000	each	X-small	100 mm	9 mm	6,5 mm	1,5 mm	14 g	11 mm
MS-001	each	small	400 mm	12 mm	8,5 mm	2 mm	83 g	19 mm
MS-002	each	medium	400 mm	16 mm	9,5 mm	2 mm	121 g	25 mm
MS-003	each	large	400 mm	19 mm	10,5 mm	2 mm	151 g	30 mm

Material specifications

- Stainless steel

Maintenance - servicing the joint

The joint is already lubricated during the assembly and the axle screw is provided with Loctite 243. When the joint components are dissembled during the production of the orthoses, they need to be greased again around the teeth and the axle screw be Loctited (243) again. We recommend checking the joints at least once every six months, or as often as the instrument maker decides in the patient’s individual situation, for functionality and wear and tear.

Package contents

- MultiStatic Corrective Joint
- MultiStatic Technical Instructions

Setting flexion / extension angle

The MultiStatic corrective Joint has an continuously angle adjustment of +120° to -120° and can be used both left and right. Note: Make sure the joint is mounted the correct way, meaning the marked arrow should be pointed up. Use an Allen Screw at the frontal side of the joint to adjust the angle of the joint. Turn the Allen screw to set the angle of the joint. The angle setting can be read on the joint’s protractor.

Aligning the joint

In order for the orthosis to function correctly, it is important the corrective Joint and Free Motion Joint are aligned correctly to one another, and a construction with sufficient torsion rigidity is built. Make sure to take the necessary precautions, and to use proper alignment tools.

Bending the joint

Proximally and distally the uprights of the joint are partly contoured (just below and above the center). Up to this point the uprights can be bend as needed to a maximum angle of 30°. Use bending tools to do so.

Cleaning instructions

The joint can be cleaned with a non-corrosive liquid substance. Try to keep the joint free from dust and dirt as much as possible.

Liability

Basko Healthcare as the manufacturer is solely liable if the product is used in accordance with the prescribed conditions and the purpose for which it is intended. Basko Healthcare recommends handling the product in accordance with applicable rules and performing maintenance as described in these instructions. Non-compliance with the above may cause the MultiStatic joints to not function or function inadequately and may impact guarantee. Changes, modifications and alterations to the product not described in these instructions are the responsibility and liability of the person making them.

CE conformity

These products meet the requirements of the MDR Ordinance (EU) 2017/745 on medical devices. These products are classified in class I (non-sterile) on the basis of the classification criteria for medical devices according to Annexes II and III of the regulation. Basko Healthcare has therefore made the declaration of conformity entirely under its own responsibility.

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