28720
THUMB ORTHOSIS

28720 S.O.T TUMORTOS

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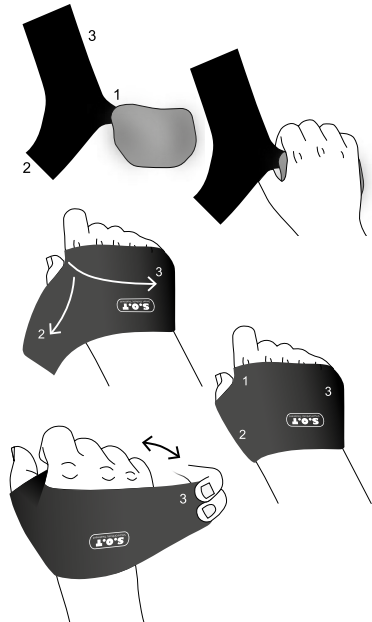
Intended use

Resting splint for the thumb designed to restrict movements in CMC I and MCP I joint and positioning the thumb in a comfortable resting position. Can be used together with a textile wrist orthosis as an extra reinforcement and support. Intended for one person use.

Indications

Osteo arthritis*, reumatoid arthritis, adduction contracture of the thumb, overload injury of the CMC-joint.

*OA in CMC region stage 1-4¹. At OA in stage 4, the scaphoid is also involved. Then the thumb support should be combined with a wrist orthosis.

¹Eaton and Littler Classification fo Basilar Thumb Arthritis.

Contraindications

No known.

Adjustments

This product should only be altered by medical professional person.

Warnings

Do not pull the straps too tight as this can cause poor circulation and/or skin irritation. Check

the skin on a daily basis and if you see or feel any irritations or other problems, remove the product immediately. If you have marks or chafing that do not reduce within a short time (30 minutes) contact your clinician immediately.

Material Specifications

This product is made from Polyester, Nylon, TPU and Spandex. Core of aluminum and polyethylene. Does not contain Latex. This product is not flame retardant.

Washing Instruction

Wash the complete product with the cover attached. Use a washing bag. Machine wash in 40°C (104°F) gentle cycle. Dry flat. Do not tumble dry or dry clean.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

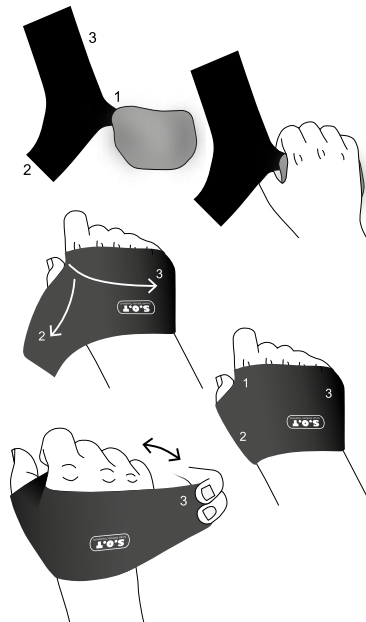
Instructions for use

WHEN FITTING

1. Check that the support is complete and without damage.
2. Open the wide straps 2 & 3 and leave strap 1 at it's position.
3. Put the hand on the support.
4. Pull the strap over the hand and attach strap 3 and then strap 2 on the Velcro on the inside of the support.
5. Adjust strap 1 and 2 for comfort.

WHEN USING

6. For donning and doffing after initial fitting, only open the strap on the little finger side (strap 3).

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