Internet: www.basko.com

E-mail: sales@basko.com


Fax: +31 (0) 75 - 612 63 73

Phone: +31 (0) 75 - 613 15 13

NL-1505 HX Zaanadam

Pieter Liefinckweg 16

Basko Healthcare

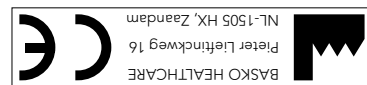Basko

Healthcare

CMCcare

Thumb brace

basko.com

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User instructions CMCcare thumb brace

Your doctor is the person who prescribed the CMCcare thumb brace and determined the treatment duration. Strict compliance with the doctor's instructions for daily wear time is necessary.

To obtain the right treatment results, it is very important to select the right size. The initial measurements and fitting of the brace must be done by professional staff. The professional who fits you the brace is responsible for the right fit and proper functioning.

In case of skin irritation, unusual pain or other complications that you think are caused by the product, you need to contact the treating physician and/or professional staff.

Intended use of the product / Indications

The CMCcare thumb brace is used for the treatment of arthrosis of the thumb basal joint caused by weakening of the joint ligaments. The CMCcare thumb brace supports the thumb basal joint (CMC-1) and keeps the metacarpal bone of the thumb (MCP-1) in a functional position.

The brace is used for the following indications:

- Arthrosis (thumb) (ICD10: M19.94)
- Instability of the thumb basal joint (CMC-1)
- Postoperative after arthroplasty of the CMC-1 joint

Additional product information / area of use:

- It is very important to check the skin at least 1x per day
- If dermatological pharmaceutical products are used on the skin underneath this product, the doctor must be consulted in advance

User risks and contraindication

The brace should not be worn on pressure points or on irritated skin. There are no other known user risks or contraindications.

Maintenance recommendation

Check the brace regularly for wear and tear and make sure that the Velcro fasteners are clean, free from dust, dirt, pieces of thread etc. to ensure optimal attachment of the straps.

Material specifications

Brace: TPU plastic; support: aluminium; strap: nylon; closure: TPU plastic; TPU-Kunststoff.

Cleaning instructions / disinfection instructions

Close all Velcro straps before washing. Hand wash at 40°C in warm water. Dry on a flat surface. Not suitable for washing machines or dryers. Do not bleach. Do not dry clean. Do not iron. Do not wring.

Instructions

Before use, check to make sure the product is undamaged and the packaging is complete.

Contents:

- 1) CMCcare thumb brace
- 2) Extra strap
- 3) Instructions for use

Fitting the brace

Step 1

First measure the circumference of the hand (without thumb) and use the size chart to select the right size (see size chart) (see Fig. 1).

Step 2

Undo the Velcro fastener on the strap, without removing it from the opening. Pull the brace down over and around the thumb so the aluminium stay fits around the thumb basal joint (see Fig. 2).

Step 3

For the best possible functionality, the aluminium stay must fit snugly around the thumb. This can be adjusted by professional staff to ensure the right individual fit (see Fig. 3).

Step 4

Now determine the correct strap length. Hold the fingers and the thumb in a curved, functional position (see Fig. 4) and close the Velcro strap. The logo on the fastener should be in the middle of the back of the hand. The strap can be shortened as needed. To do so, remove the Velcro fastener from the strap and use scissors to shorten it to the desired length (see Fig. 5).

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