Pavlik

Hip abduction brace

Instructions

Basko Healthcare

Pieter Liefinckweg 16 | NL-1505 HX Zaandam
Phone: +31 (0) 75 - 613 15 13 | Fax: +31 (0) 75 - 612 63 73
E-mail: verkoop@basko.com | Internet: www.basko.com

basko.com

000000 / Stand: 01/2018

Basko
Healthcare
basko.com

Size chart

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Pavlik Hip abduction brace

User instructions

The treating physician is the person who prescribes the harness and determines the duration of the treatment. The physician's instructions about daily wear times must be followed strictly and carefully.

Selecting the right size is important for optimal treatment, so the first fitting with the orthosis must be done by a professional. The person who does the fitting is responsible for selecting the right settings and making sure that the harness works properly. The parents/caregivers must also receive careful instruction about how to put the harness on and take it off. It should also be stressed that the settings should never be changed by the parents/caregivers unless specified otherwise by the treating physician.

In case of skin irritation, unusual pain or other complications that you think are caused by the product, you must contact the treating physician and/or technical staff immediately.

Intended use of the product / Indications

The Pavlik harness is used for the treatment of disorders of the hip joint. The legs are positioned in such a way that the hip socket stays anchored in place so it can mature and hip (sub)luxation is prevented.

The harness is used for the following indications:

- Hip dysplasia (ICD10: Q65.8)
- Hip luxation (ICD10: S73.00)
- Hip subluxation, congenital (ICD10: Q65.5)

Additional product information / areas of use

Skin care:

- It is very important to check the condition of the skin at least once a day, in particular the shoulders, neck, back, chest, stomach, knee creases, legs and ankles.
- Medicinal ointments should be used only after consultation with the treating physician.
- For the sake of comfort and hygiene, the harness should always be worn over clothing, and socks must be worn as well.

Bath advice:

- Bathe the baby as quickly as possible in order to keep the time out of the harness to a minimum.
- It is a good idea to gently massage the contact points of the body with the harness to help the baby's muscles relax.
- The harness is not designed for use in salt water.

Risks associated with use and contraindications

Do not use on pressure points or on areas with irritated skin. There are no other known contraindications or risks associated with use.

Maintenance recommendation

Check the harness regularly for wear and make sure that the Velcro straps are free from dust, dirt, bits of thread etc. so the straps are always fastened securely.

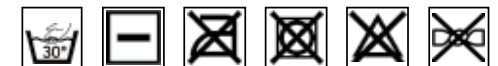
Material specifications

Harness: nylon; lining: PU foam

Cleaning instructions / disinfection instructions

Close all Velcro straps before washing. Hand wash at 30° C in warm water. Let it air dry on a flat surface. Not suitable for washing machine or dryer. Do not bleach or dry clean. Do not iron.

Recommendation: after washing, wrap the product in a towel to remove excess water. Do not wring.

Instructions

Check the product for any damage before use and make sure the content of the package is complete. Contents: **1)** Pavlik harness

2) User instructions

Applying the harness

Measure the chest circumference and refer to the size chart to find the correct size (see size chart).

Step 1

Measure the length of the foot and cut the liner of the bootie along the dotted line(s) to achieve the best fit (Fig. 1). Reattach the liner to the harness. Repeat this step for the other side.

Step 2

Undo the chest strap and position the baby so the midline of the back is where the straps cross (Fig. 2).

Step 3

Guide the arm through the shoulder strap (Fig. 3), and repeat this step on the other side.

Step 4

Close the chest strap at the front, making sure it's positioned just below the axilla (armpit), the shoulder straps are adjusted to the correct length and are resting midpoint on the shoulders (Fig. 4). Adjust if necessary.

Step 5

Wrap the bootie liner around the foot; close the lower ankle strap first. Across the front from medial to lateral and attach the Velcro strap. Next, close the upper ankle strap across the back from medial to lateral and attach the Velcro strap (Fig. 5). Repeat these steps on the other side.

Step 6

The leg and foot straps have a corresponding color coding. Always ensure that color closes by color. Red and Blue are the flexion straps and Green and Yellow are the abduction straps. Adjust the flexion straps making sure the legs are 90° flexed (Fig. 6).

Step 7

Now adjust the abduction straps with the leg in 60° abduction (Fig. 7). Repeat this step on the other side. Flexion and abduction settings are just a general guideline. Any instructions from the physician that deviates from this should always be respected.

Step 8

Continue to check all straps once more for correct positioning (Fig. 8). We recommend using a pen to mark all the closure setting on the straps making sure they are positioned correctly next time applied.

Taking the harness off

Note: you do NOT need to take the harness off to change the baby's diaper. Bathe the baby as quickly as possible in order to keep the time out of the orthosis to a minimum.

To ensure that the flexion settings are not changed when taking off the harness, simply undo the two ankle straps to take the feet out of the harness. Then undo the chest strap and pull the arms through the shoulder straps to remove the rest of the harness.

To put the harness back on, pull the arms through the shoulder straps again and reattach the ankle straps in accordance with the instructions in Step 6.

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