Club foot care for children

Information for parents/carers

The development of your child

Your child will not experience any hindrance to his/her development due to the orthosis. Many children are able to get used to the orthosis quickly.

The adjustment of the orthosis

In the beginning, the orthosis has been normally set so that the foot concerned points 70 degrees outwards. For a healthy foot, the angle is 40 degrees. The positions of the feet may possibly seem extreme for someone not familiar with the Ponseti method, but these are normal settings for proper treatment. The feet were also in this position during the treatment with plaster casts.

What problems might arise?

Redness and pressure sores: your child's skin is perhaps not yet ready for the wearing of shoes and is still vulnerable. Although the Clubycare shoes are made of soft and supple leather, pressure sores or blisters may sometimes appear. It is then extremely important that the foot is properly and firmly in place in the shoe. If the foot can slide backwards and forwards in the shoe, pressure sores or blisters may arise. When you see red, irritated places after removing the shoes, first see whether these disappear within a few minutes. Should they persist or gradually get worse, contact your supplier or the treating doctor. The treating doctor will then determine further treatment.

Some final advice

Shoes: You must be aware that the shoes are not normal shoes for wearing, but special orthosis shoes. Due to the rigid, inflexible sole, specially developed for this therapy, you child is not able to set his or her foot down properly while walking. This means that the sole does not have any grip on a smooth surface. Without the splint, the shoes also have no therapeutic value whatsoever. You child must therefore only wear the shoes in combination with the splint. The **standard** Clubycare shoes do not have left and right versions. Sometimes, for accessibility, depending on the position of the shoe, it may be easier when the buckle of the strap is placed on the inside of the foot instead of, as normal, on the outside.

Clothing: Have your child wear socks without patterns, thick stitching or labels. These types of socks can, in particular, cause pressure sores since they are of uneven thickness. The socks should also not be too big since folds may occur that could give rise to pressure sores. The wearing of a sleep suit is not a problem.

Protection: Depending on how lively and active your child is, you or your child may accidentally damage the orthosis. The P.E.A.care Splint also have a completely smooth underside, so when your child starts to crawl or stand in the orthosis, your floor or furniture may be damaged.

Seats and prams: All official baby carriers are suitable for the wearing of the orthosis. Normally speaking, a buggy or pram is not a problem. Only in the choice of a high chair or swing must you take account of the fact that your child is able to sit in them without loosening the shoes.

Bedtime: Your child may be woken in the night because the orthosis has become entangled in the bedding. We advise having your child sleep in a sleeping bag. This means that wearing the orthosis becomes part of 'going to bed'. Before you know it, your child will be asking for the orthosis when he or she feels tired! To become familiar with the wearing of the orthosis, keeping a diary may help. In this diary, note each day how long, for example, your child has worn the orthosis. The initial requirement is that your child wears the orthosis for 23 hours a day for the first 3 months. After this period, in discussion with the treating doctor, a reduction may be considered.

We wish you every success and perseverance with using the orthosis as prescribed. Should you still unfortunately encounter problems, contact your orthotist.

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