
Interne: www.basko.com
E-mail: sales@basko.com
Fax: +31 (0) 75 - 612 63 73
Phone: +31 (0) 75 - 613 15 13
NL-1505 HX Zaandam
Pietert Liefthinkweg 16
NL-1505 HX Zaandam
Pietert Liefthinkweg 16

Basko Health care

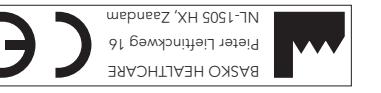
basko.com

Basko
Health care

MultiMotion

Alignment Unit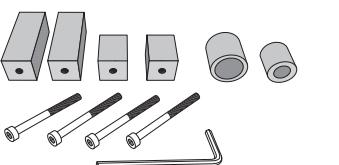

NL	Gebruiksaanwijzing
DE	Gebrauchsanleitung
ENG	Instructions for Use
FR	Mode d'emploi
ES	Uso y aplicación
IT	Usi e applicazioni
SK	Bruksanvisning
DK	Bruksanvisning
NO	Bruksanvisning
FI	Sovitusohjeet
CZ	Návod k použití
SI	Navodila za uporabo

Basko
Health care
basko.com

NL

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ENG

Application Guidelines

Proper construction with associated joint alignment is critical to a properly functioning orthosis. This alignment kit facilitates joint alignment during the production of an orthosis with the MultiMotion joint. This alignment kit can also be used in conjunction with a FreeMotion joint, if applicable for which we recommend the additional AK-104 alignment kit.

The certified technicians producing the orthosis are responsible for the production technique chosen and the correct functioning of the orthosis. Our advice is to use the product-specific alignment unit at all times during production.

Prior to production, all possible openings on the joint should be sealed with suitable material to prevent the joint from filling up with resin or similar.

Additional product information
The alignment unit is suitable for multiple use.

Maintenance instructions
After use, inspect that all parts are undamaged, clean and free of plaster and similar.

Instructions
The alignment unit for the MultiMotion joint consists of 2 separate parts:

AK-100 Basic T-Bar # AK-102 MM Kit

First check that all parts shown in Figure 1 are complete and undamaged in the package:

Package contents AK-100:

A) 1 x T-Bar

Package contents AK-102:

- A) 2 x long bar profile
- B) 2 x short bar profile
- C) 1 x round profile large
- D) 1 x round profile small
- E) 4 x cylindrical screw with hexagon socket
- F) 1 x Allen key (3 mm)
- G) Instruction for use

Step 1: While casting the patient, mark the joint axes. After removing and sealing the negative, perforate the axis markings at the desired position and then insert the T-bar through the perforation. After piercing, check that the axis is horizontal and close the negative around the T-bar (Fig. 2).

Step 2: Fill the negative with plaster possibly in combination with Vermiculite (Art. no. 17) and let it harden well. After curing, remove the plaster negative and the T-bar. Then correct the plaster negative as needed and allow it to dry (Fig. 3).

Step 3: Use the supplied 3 mm Allen key to loosen the central screw on the top of the joint (Fig. 4).

Step 4: Then remove in sequence the top cover, smaller plastic ring, indicator plate, spring housing and the larger plastic ring (Fig. 5).

Step 5: Then, depending on the model, place the large or small round profile on the top of the joint. Now position a hexagon socket cylinder screw at the front through the round profile and the central axis of the joint (Fig. 6).

Step 6: Then screw (depending on the M-L width of the plaster) the short or long bar profile onto the cylinder head screw with the supplied Allen key (Fig. 7).

Step 7: Then laterally place the MultiMotion joint parallel to the "joint" using the bar profile in the opening where the T-bar has been (Fig. 8).

Step 8: Where necessary, adjust the bars to the contours of the plaster cast (Fig. 9).

For a bilateral orthosis assembly (with the FreeMotion joint as a companion joint), follow the instruction as described in the FreeMotion Alignment Unit (# AK-104) Instructions for Use.

After production, remove the alignment unit by following steps 3 through 8 in reverse order, and follow the instructions in the joint's user manual to adjust and check for proper function.

The alignment unit is suitable for multiple use.

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