
Internet: www.basko.com
 E-mail: sales@basko.com
 Fax: +31 (0) 75 - 612 63 73
 Phone: +31 (0) 75 - 613 15 13
 NL-1505 HX Zaandam
 Pieter Lieftrinkweg 16

Basko HealthCare


basko.com

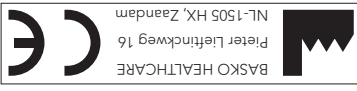MultiStatic

Alignment Unit

AK-103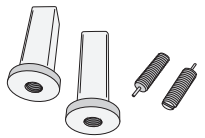

- NL Gebruiksaanwijzing
- DE Gebrauchsanleitung
- ENG Instructions for Use
- FR Utilisation et application
- ES Uso y aplicación
- IT Usi e applicazioni
- SK Bruksanvisning
- DK Bruksanvisning
- NO Bruksanvisning
- FI Sovitusohjeet
- CZ Návod k použití
- SI Navodila za uporabo

NL

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ENG

Application Guidelines

Proper construction with associated joint alignment is critical to a properly functioning orthosis. This alignment kit facilitates joint alignment during the production of an orthosis with the MultiStatic joint. This alignment kit can also be used in conjunction with a FreeMotion joint, if applicable for which we recommend the additional AK-104 alignment kit.

The certified technicians producing the orthosis are responsible for the production technique chosen and the correct functioning of the orthosis. Our advice is to use the product-specific alignment unit at all times during production.

Prior to production, all possible openings on the joint should be sealed with suitable material to prevent the joint from filling up with resin or similar.

Additional product information

The alignment unit is suitable for multiple use.

Maintenance instructions

After use, inspect that all parts are undamaged, clean and free of plaster and similar.

Instructions

The alignment unit for the MultiStatic joint consists of 2 separate parts:

- | | |
|-----------------------------|------------------------|
| # AK-100 Basic T-Bar | # AK-103 MS kit |
|-----------------------------|------------------------|

First check that all parts shown in Figure 1 are complete and undamaged in the package:

Package contents AK-100:

- A)** 1 x T-bar

Package contents AK-103:

- A)** 2 x bar profile with screw holder
B) 2 x threaded adapter
C) Instruction for use

Step 1: While casting the patient, mark the joint axes. After removing and resealing the negative, perforate the axis markings at the desired position and then insert the T-bar through the perforation. After piercing, check that the axis is horizontal and close the negative around the T-bar (Fig. 2).

Step 2: Fill the negative with plaster possibly in combination with Vermiculite (art. no. 17) and let it harden well. After curing, remove the plaster negative and the T-bar. Then correct the plaster negative as needed and allow it to dry (Fig. 3).

Step 3: Using the bar profile, place the joint parallel to the "joint" in the square T-bar opening (Fig. 4).

Step 4: Then screw the threaded adapter with the blunt side into the bar profile. Turn it in until the desired offset is achieved (Fig. 5).

Step 5: The pin of the threaded adapter fits into the opening on the back of the MultiStatic joint (Fig. 6).

Step 6: Where necessary, bend the rods to the contours of the plaster positive (Fig. 7).

For bilateral orthosis assembly (with the FreeMotion joint as a companion joint), follow the instructions as described in the user manual for the FreeMotion alignment unit AK-104. After assembling the orthosis, remove the bar profile with threaded adapter.

The alignment unit is suitable for multiple use.

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