**28720**  
**THUMB ORTHOSIS**

## 28720 S.O.T TUMORTOS

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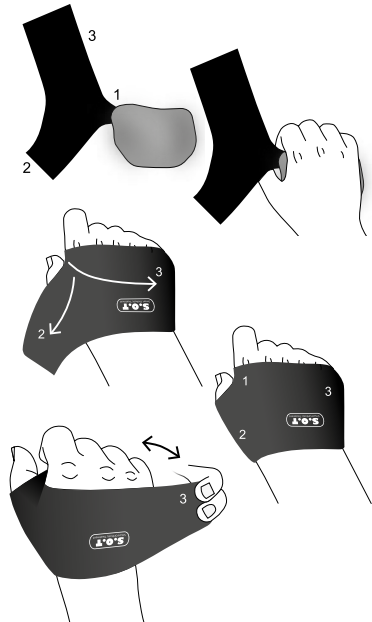
### Intended use

Resting splint for the thumb designed to restrict movements in CMC I and MCP I joint and positioning the thumb in a comfortable resting position. Can be used together with a textile wrist orthosis as an extra reinforcement and support. Intended for one person use.

### Indications

Osteo arthritis\*, reumatoid arthritis, adduction contracture of the thumb, overload injury of the CMC-joint.

\*OA in CMC region stage 1-4<sup>1</sup>. At OA in stage 4, the scaphoid is also involved. Then the thumb support should be combined with a wrist orthosis.

<sup>1</sup>Eaton and Littler Classification fo Basilar Thumb Arthritis.

### Contraindications

No known.

### Adjustments

This product should only be altered by medical professional person.

### Warnings

Do not pull the straps too tight as this can cause poor circulation and/or skin irritation. Check

the skin on a daily basis and if you see or feel any irritations or other problems, remove the product immediately. If you have marks or chafing that do not reduce within a short time (30 minutes) contact your clinician immediately.

### Material Specifications

This product is made from Polyester, Nylon, TPU and Spandex. Core of aluminum and polyethylene. Does not contain Latex. This product is not flame retardant.

### Washing Instruction

Wash the complete product with the cover attached. Use a washing bag. Machine wash in 40°C (104°F) gentle cycle. Dry flat. Do not tumble dry or dry clean.

### Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

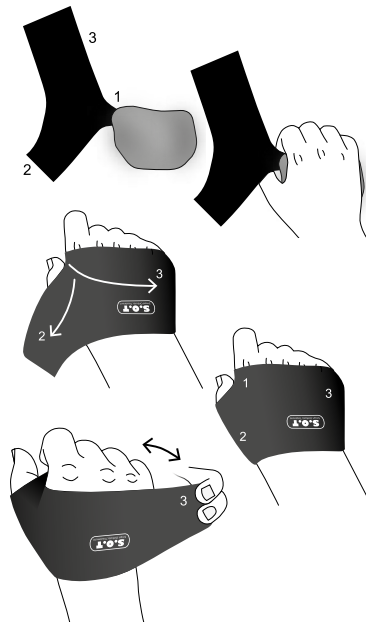
## Instructions for use

### WHEN FITTING

1. Check that the support is complete and without damage.
2. Open the wide straps 2 & 3 and leave strap 1 at it's position.
3. Put the hand on the support.
4. Pull the strap over the hand and attach strap 3 and then strap 2 on the Velcro on the inside of the support.
5. Adjust strap 1 and 2 for comfort.

### WHEN USING

6. For donning and doffing after initial fitting, only open the strap on the little finger side (strap 3).

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