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8403301 / Stand: 11/2017

User instructions for semi-finished Stomacare belt

Contents of the package of articles 301, 304 and 305:

1. A stoma belt that is unfinished at the top or bottom, thus allowing you to choose whether to use a base plate in front of the stoma opening or not.
2. Elastic, fitted with Velcro tape (anti-prolapse cover)
3. Elastic trim with a folding edge in the middle
4. Instructions and ruler
5. User instructions for applying the stoma pouch

Instructions

- Place the belt on the patient, at the desired tension
- Use a soft pencil to mark the centre of the stoma
- Place the belt flat on a table and use a soft pencil and the ruler at the lower end of this card to draw the desired circle. (**N.B.** Make sure the circumference is just large enough to allow insertion of the outer ring of the stoma system, + 0.5 cm to allow room for the elastic trim)
- To prevent the various layers (2x material and a possible base plate) from slipping, the part around the circumference can be stitched down
- Fold the circle you have drawn in two and make a tiny cut in it
- Cut the whole circle out
- Stitch the elastic trim with the pre-pressed fold to the inside of the belt, around the opening of the stoma (take into account the thickness of the material being used!)
- Fold the elastic through the opening to the outside of the belt and again stitch it into place
- Stitch the unfinished elastic to the top or bottom of the stoma belt
- Stitch the elastic with Velcro tape to the belt in order to seal the opening of the stoma and prevent a prolapse and/or to keep the stoma pouch in position

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