## FR

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## ENG

**Description and intended use:**

Made of a soft material that adjusts perfectly to your body. Comfortable, lightweight and snug. Provides a light degree of pressure that is useful for mild to moderate pain, for long-term treatment or as a preventative measure for activities that place stress on the lower back.

**Indications:**
**PRS600 - PRS601:** Lower back pain. Lower back discomfort that requires the use of heat. Lumbar support for post-operative and/or post-partum care.
**PRS606:** Mild or moderate lower back pain. Lumbar support for post-operative and post-partum care. Muscle weakness. Degenerative processes.
**PRS607:** Mild or moderate lower back pain. Post minor trauma. For preventative care during work or sporting activities involving strain on the lower back region.

- Precautions:**
- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop where you purchased it.
  - The general instructions of use and specific indications determined by the doctor should always be followed.
  - Only a doctor is qualified to prescribe treatment and determine its duration and monitoring.
  - If you notice any side effects, skin conditions or sensitisation, consult your doctor immediately.
  - Your doctor might consider the use of topical creams in conjunction with the orthosis.
  - In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
  - Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
  - The first fitting of the orthosis should be carried out either by qualified personnel (doctor, orthopaedic technician, etc.) or under their supervision.
  - For the orthosis to function properly, its components must be regularly checked to ensure that they are in good condition. The medical staff responsible for monitoring of the patient's treatment may explain the suitability of the product to the patient or suggest its replacement if it is damaged or worn.
  - Make sure that the quick Velcro fasteners are free from fluff and lint to ensure that their closure properties are preserved.
  - Clean the product regularly and maintain proper hygiene.
  - Do not wear the orthosis while having a bath, shower, etc.
  - Do not let children play with this device.

**Instructions for proper fitting:**

**PRS600:** Insert your feet into the girdle as shown in (Fig. A) and pull it up until it is approximately 5 to 7 cm above the beginning of the gluteal fold (Fig. B).
**PRS601:** Position the girdle in the centre of the back as shown in (Fig. C), placing it approximately 5 to 7 cm above the beginning of the gluteal fold and secure it using the Velcro fasteners (Fig. D).
**PRS606:** Position the girdle by positioning the stays on either side of the spine as shown in (Fig. E), placing it approximately 5 to 7 cm above the beginning of the gluteal fold, and secure it using the Velcro fastener (Fig. F1 - F2).
**PRS607:** Position the girdle by positioning the stays on the back as shown in (Fig. G), placing it approximately 5 to 7 cm above the beginning of the gluteal fold and secure it using the Velcro fastener (Fig. H1 - H2).

**Product maintenance - washing instructions:**

Hand wash. Do not use bleach. Do not tumble dry. Do not iron. Girdles with Velcro fasteners: Close all of the Velcro fasteners before washing.

**Note:** Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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